



HEALTH ADVISORY

May 3, 2024

NOTICE OF AUTOMATED EXTERNAL DEFIBRILLATOR PAD INCOMPATIBILITY

Guidance for all healthcare facilities, emergency medical services agencies, and public access defibrillation providers utilizing automated external defibrillators.

The New York State Department of Health Emergency Medical Services for Children Advisory Committee has recommended that a health advisory, based on the National Association of State EMS Officials memorandum from January 25, 2022¹, be transmitted to all healthcare providers, emergency medical services agencies, and public access defibrillation providers utilizing automated external defibrillators. The memorandum outlines a critical issue of Automated External Defibrillator (AED) pad incompatibility affecting the efficacy of pediatric emergency care.

Key Points:

1. **Zoll Pedi-pads:** These defibrillation pads are not compatible with Zoll AED Plus units². The compatible defibrillation pads for these units are the Zoll Pedi-pads II.
2. **Physio-Control LIFEPAK 15:** Non-manufacturer recommended defibrillation pads do not support the necessary therapy for certain age and weight ranges and should not be used with LP-15 units for pediatric patients. As specified in the LP15 operating instructions, AED mode is intended for use on patient eight years of age and older³.
3. **General Advisory:** Some AEDs are accompanied by pediatric pads that are not suitable for AED use. These are usually marked with specific warnings on the packaging.

Action Required:

All healthcare providers, emergency medical services agencies, and public access defibrillation providers utilizing automated external defibrillators should ensure all AED units have the correct, manufacturer-recommended pads and that all operators of AED devices are aware of these incompatibilities.

Further Information:

Refer to the attached National Association of State EMS Officials Defibrillator Pad Incompatibility Memo and the manufacturer's instructions for detailed guidelines. For additional questions, contact Amy Eisenhauer, Emergency Medical Services for Children Program Manager at amy.eisenhauer@health.ny.gov, or (518) 485-5907.

¹ National Association of EMS Officials Memorandum:

<https://nasemso.org/wp-content/uploads/Defibrillator-Pad-Incompatibility-Memo-2022-0125.pdf>

² Zoll AED Plus Administrators Guide, Page 30:

https://www.zoll.com/-/media/product-manuals/aed-plus/01-english/9650-0301-01-sf_yd.ashx

³ Lifepak 15 Operating Instructions Manual, Page 11:

https://www.stryker.com/content/dam/stryker/ems/resources/operating-instructions/international/3314911-030_int-eng_lifepak_15_operating_instructions.pdf



National Association of State EMS Officials

201 Park Washington Court • Falls Church, VA 22046-4527 • www.nasemso.org
703-538-1799 • fax 703-241-5603 • info@nasemso.org

MEMORANDUM

TO: NASEMSO Members
FROM: NASEMSO Pediatric Emergency Care Council
SUBJECT: Defibrillator Pad Incompatibility
DATE: January 25, 2022

This information was brought to the Board of Directors who supported its distribution to members in the interest of protecting the public.

Situation:

The NASEMSO Pediatric Emergency Care (PEC) Council recently learned of several issues regarding automatic external defibrillator (AED) pad incompatibility, with multi-function defibrillator and AED pads not being used in accordance with manufacturer's recommendations, particularly in pediatric applications.

Background:

In recent inspections, it was identified that some EMS services and/or their personnel are utilizing incorrect pediatric defibrillator pads with various manual monitor/defibrillators and AEDs. This results in the inability to deliver therapy across all age and weight ranges. On November 29, 2021, the Massachusetts Office of EMS issued a statewide memorandum regarding this issue and shared it with the NASEMSO PEC Council. Subsequently, this has been found to be occurring in several other states as well.

Assessment:

Examples include:

- Zoll Pedi-padz: These defibrillator pads, although manufactured by Zoll, are not compatible with the Zoll AED Plus units and will not plug into the unit. The pediatric pads required for the Zoll AED Plus units are the Pedi-padz II. The Zoll AED Plus Administrators Guide, page 30, states to use "ZOLL Stat-padz II, CPR-D-padz or Pedi-padz II" and to "Only use electrodes labeled "Infant/Child" on children less than 8 years old or weighing less than 55 lbs. (25 kg). Use CPR-D-padz® if patient is older than 8 years or weighs more than 55 lbs. (25 kg)."

- Physio-Control Lifepak-15: Use of non-manufacturer recommended pads for LP-15 units, with the intent of using them for pediatric patients. These pads have therapy limitations for both age and weight ranges for infants and children in both the manual and AED mode on the LP-15. The pads' packaging states that in manual defibrillator mode, these pads cannot be used for patients weighing under 22 pounds, and in AED mode, are not to be used for patients under age 8. As a reminder, providers cannot use an LP15 in the AED mode for pediatrics, regardless of pad type. The LP15 operator's manual, in section 5 pages 5-7, under "Automated External Defibrillation (AED)" states, "In AED mode, the LIFEPAK 15 monitor/defibrillator is not intended for use on pediatric patients less than 8 years old."
- General: Some AEDs have been identified that were accompanied by pediatric pads with packaging that had red writing and a circle with a line through it around an AED symbol that stated, "not for AED use."

Recommendation:

Although pediatric defibrillation is an uncommon event, ensuring that properly compatible equipment is available is one of the keys to this life-saving intervention. We are recommending that state and regional EMS leadership share this information and remind all EMS, Fire, and other public safety agencies to double-check all monitors and AED units to ensure they have the correct manufacturer recommended pads stocked in all units. For questions, please refer to the manufacturer instructions for the device, contact the manufacturer, and/or contact state or regional EMS leadership.

References

1. Lifepak-15 Operators Manual https://www.physio-control.com/uploadedFiles/Physio85/Contents/Emergency_Medical_Care/Products/Operating_Instructions/LIFEPAK15_OperatingInstructions_3306222-002.pdf
2. Zoll AED Plus Administrator's Guide https://www.zoll.com/-/media/product-manuals/aed-plus/01-english/9650-0301-01-sf_yd.ashx
3. Massachusetts Office of EMS Memorandum https://www.cmesc.org/images/Announcements/Memorandum_Ensuring_Defibrillator_Pad_Compatibility_to_Provide_Effectivepdf.pdf